

The Undergarment or Artificial Foreskin for Circumcised Men



INSTRUCTIONS FOR USE

The following instructions should be read carefully before using the **SenSlip**

How the **SenSlip** Works
Fitting & Wearing Your **SenSlip**
Looking After Your **SenSlip**

How the **SenSlip** Works

The **SenSlip** Undergarment has been designed as an Artificial or Synthetic Foreskin for the circumcised penis.

When a penis is circumcised the protective foreskin is removed leaving the glans or head of the penis exposed. The glans is designed to be moist and very sensitive to touch and feel. When the foreskin is removed some functions of the penis are lost forever.

The surface membrane of the glans becomes desensitized and thickens by up to twelve times its normal thickness. This process is called 'keratinization'.

Although the **SenSlip** can never replace a real foreskin it does act as a protective cover to the glans and like a natural foreskin prevents chaffing and rubbing against clothing.

When the **SenSlip** is worn, immediate protection is provided and over time the natural process of 'de-keratinization' and re-sensitization of the glans begins. An increase in sensitivity can be achieved within ten to fourteen days.

Caution

This product contains natural rubber latex which may cause allergic reactions. Safe use of this product by or on latex-sensitive individuals has not been established.

The **SenSlip** Undergarment is not suitable for males under the age of eighteen years.

For further information on the **SenSlip** product please visit our FAQ website: www.viafin-atlas.com
or email us at www.info@viafin-atlas.com



Units No: 15 & 16 Lancaster Road
Sarum Business Park
Salisbury SP4 6EA
Wiltshire
United Kingdom.

www.viafin-atlas.com

The **SenSlip** can be adjusted for different lengths of penis. This is discussed on the next page.

"The vast majority of the world's adult male population are intact and for all the right reasons wish to keep the foreskin that they were born with". They know.

